

Binocular Autorefractor



reddot award 2014
winner



Short Manual 6

“Documenting measurement results electronically”

Plusoptix GmbH
Neumeyerstrasse 48
90411 Nuremberg
Germany
www.plusoptix.com

Table of Contents

1	Documenting measurement results – an overview.....	3
2	Documenting the measurement results electronically	3
2.1	Saving measurement reports	4
2.2	Adding an individual infographic.....	5
2.3	Accessing the measurement report.....	5
3	Full integration with your EMR system (only A12C and A16).....	7

List of figures

Figure 1:	Measurement report (Letter-sized or DinA4).....	5
Figure 2:	Manual connection of the storage device to a workplace computer	6
Figure 3:	Manual LAN / WLAN network access	7
Figure 4:	Automated connection with the freeware plusoptiXconnect	7
Figure 5:	Automated connection to your Electronic Medical software	8
Figure 6:	Automated transfer of patient data and measurement results.....	8



This manual is a supplementary short manual. For important information about your Plusoptix device, such as its intended use, basic handling, technical information, maintenance, service, and warranty information, please refer to the user manual that was delivered with your device. This user manual can also be downloaded on our homepage (plusoptix.com), under support.

List of further supplementary short manuals

1. Checking the scope of delivery and learning about the device
2. Adjusting settings and setting up WLAN
3. Preparing and performing a measurement
4. Viewing the measurement results
5. Entering, retrieving or deleting patient data
6. Documenting measurement results electronically
7. Documenting measurement results on paper
8. Exporting back-ups and reports (only plusoptiX A12C and A16)
9. Downloading software updates
10. Troubleshooting malfunctions and measurement interruptions

1 Documenting measurement results – an overview

The measurement results can be documented electronically or on paper.

Electronic documentation

Documenting electronically means that you can save measurement reports (see figure 1) on your workplace computer and also attach them manually to your electronic patient records.

If the device is connected to your Electronic Medical Record system for csv data exchange, the measurement report and the measurement values can even be saved automatically to your Electronic Medical Record system (refer to chapter 2.3 “Accessing the measurement report”).

Documentation on paper

Documenting on paper means that the measurement results can be printed out to attach them to a patient paper record. You can print a self-adhesive label on the wireless infrared label printer “plusoptix P12” or a measurement report on any printer that is connected to your workplace computer. With the “plusoptiX A12C / A12R” model, it is also possible to print the measurement report directly on a mobile USB printer, without using a workplace computer.

More information about the documenting the measurement results on paper can be found in short manual 7.

2 Documenting the measurement results electronically

Touch the printer (1) button in the navigation bar of a results page to do this.

In the “plusoptiX A12C” and “plusoptiX A16” model, the patient data input screen is then displayed. Enter the patient data (see supplementary short manual 5), which should be displayed on the measurement report and then touch the green checkmark (2) to confirm.

If the measurement results are to be documented anonymously, touch the checkmark (2) in the navigation bar without entering patient data. This will allow you to skip the patient data entry screen. The device then saves the measurement under a consecutive patient ID. This is composed as follows:

<last 4 digits of the serial number-><consecutive number>-<date>-<time>



Note:

The “plusoptiX A12R” model does not have an internal patient database. Touching the printer button after the measurement in the “plusoptiX A12R” model will display a single line to enter individual patient data. This data will be portrayed on the measurement report and on the self-adhesive label.

After confirming the patient data entry or the anonymous, consecutive patient ID (see supplementary short manual 4), a print job for label printing is sent via the infrared interface and a measurement report is saved to the inserted SD card (plusoptiX A12) or USB stick (plusoptiX 16) at the same time.



2.1 Saving measurement reports



Attention:

Measurement reports can no longer be called up in the internal patient database at a later point in time. Save the measurement report by adding it to your patient record or transferring it to your network computer (see 2.3 “Accessing the measurement report”).



Attention:

Saving the measurement report takes a few seconds and depends on the storage medium. If the SD card or USB stick is removed too soon or the URL is called up too early, the PDF file will not be saved.

The SD card or USB stick must then be reinserted in the device in order to save the measurement report again on the SD card or USB stick. If the URL was called up too early, it must be called up again.

The measurement report is used to document the measurement values and the last camera image. Depending on the time of entering patient data, the measurement report is saved automatically or manually:

Time of entry	Storage of the measurement report
Entry or Retrieval of patient data before the measurement (only A12C und A16)	The measurement report is automatically saved as a PDF file at the end of every successful measurement. If you see the status message “Try again”, the measurement report must be saved manually.
Entry or Retrieval of patient data after the measurement (only A12C und A16)	After the confirmation of patient data, the measurement report is automatically saved as a PDF file at the end of every successful measurement.
No entry of patient data, documentation via patient ID number	The measurement report is saved after entering or retrieving the patient ID as well as after confirmation of the anonymous, consecutive patient ID number.

To be able to allocate these measurement reports to individual patients, the entered patient information can be found in the file names. In addition to the entered data, a date and time stamp and the measurement result are added.

PDF file name:

Last name_FirstName-YYYYMMDD-HHMMSS.pdf

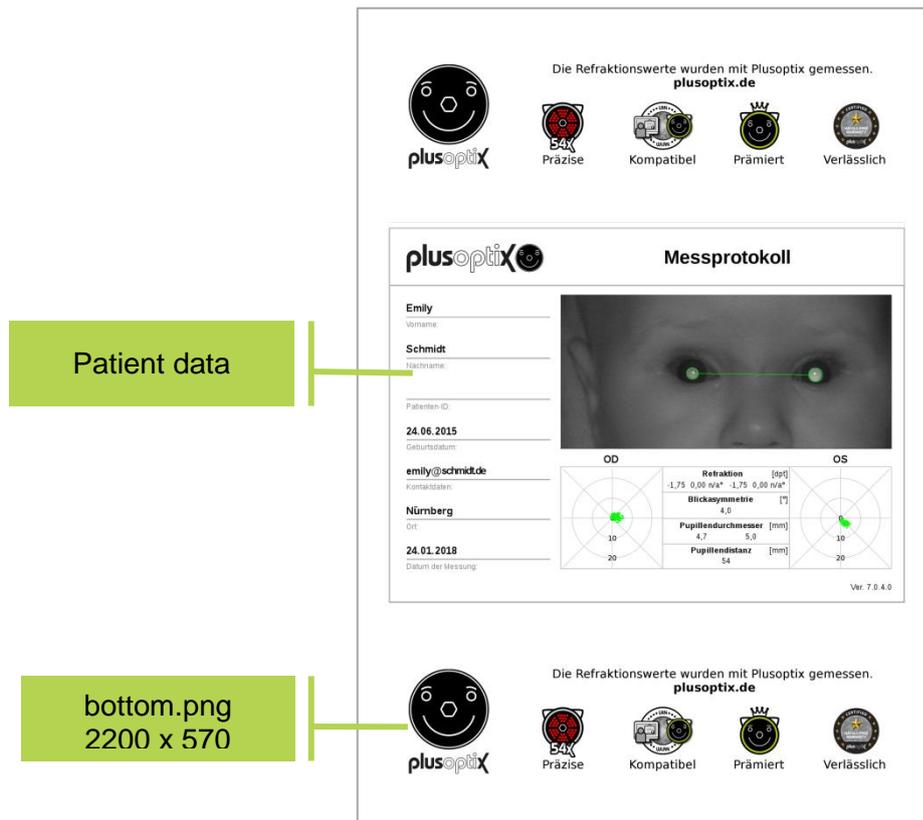


Figure 1: Measurement report (Letter-sized or DinA4)



Note:

If a patient ID has been entered instead of or as well as the patient name, this patient ID will be visible in the file name instead of the patient name

2.2 Adding an individual infographic

Infographics are found on the upper and lower edge of the measurement report. You can design them as you wish. Create your graphic in png format with a size of 2200 x 570 pixels (w x h) and name it "top" for the upper edge and "bottom" for the lower edge of the measurement report. Copy your graphic into the main directory of a USB stick. As soon as you connect the USB stick to the operational device your graphic will be uploaded automatically. You can find a sample graphic with the correct measurements for the information column in the download area on our homepage (plusoptix.com).

2.3 Accessing the measurement report

After saving the measurement report to the SD card (plusoptiX 12) or USB stick (plusoptiX 16), there are three different options to electronically access them:

- Manual connection of the SD card or USB stick to a workplace computer
- Manual LAN / WLAN network access (only A12C and A16)
- Automated connection with the freeware plusoptiXconnect (only A12C and A16)

Following, please find detailed explanations.

a) Manual connection of the SD card or USB stick to a workplace computer

Connect the SD card (plusoptiX A12C / A12R) or USB stick (plusoptiX A16) to a workplace computer. The storage will be displayed as “removable medium” in your Windows Explorer or Apple Finder.

Open the folder “pdf” and choose the measurement report you want save. You can copy it to any folder on your workplace computer or in the network.

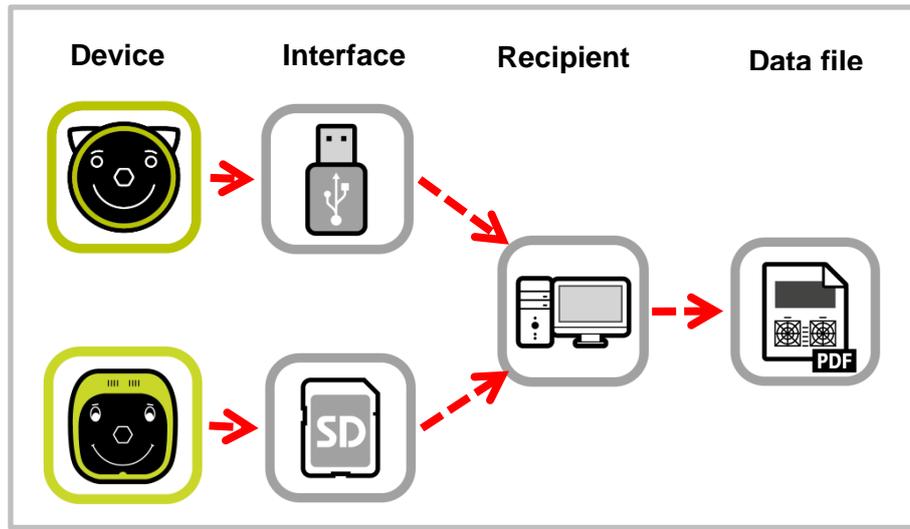


Figure 2: Manual connection of the storage device to a workplace computer

b) Manual LAN / WLAN network access (only A12C and A16)

Open your Windows Explorer or Apple Finder. You can access the measurement report at following URLs:

plusoptiX A12C: \\px12-xxxx\pdf (Windows Explorer)
 (WLAN) smb://px12-xxxx/pdf (Apple Finder)

plusoptiX A16: \\px16-xxxx\pdf (Windows Explorer)
 (LAN and WLAN) smb://px16-xxxx/pdf (Apple Finder)

The “xxx” place marker stands for the last four numbers of the serial number.

Choose the measurement report you want save. You can copy it to any folder on your workplace computer or in the network.



Note:

If the workplace computer displays the error message “Access denied”, restart Windows Explorer or Apple Finder.

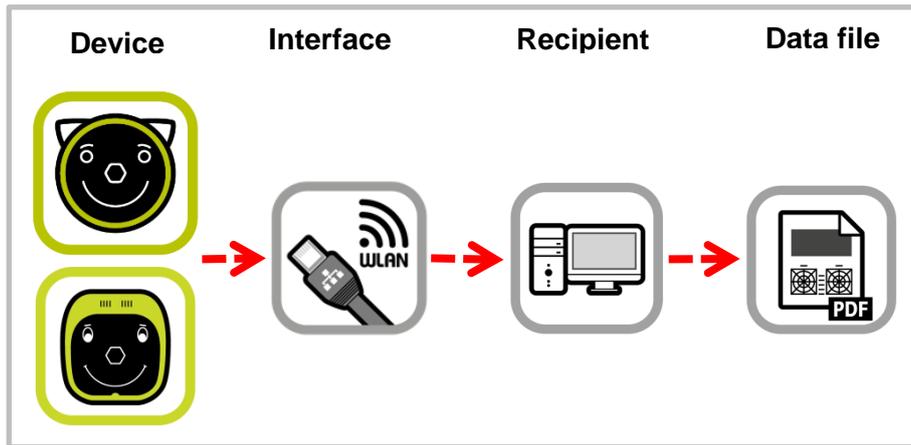


Figure 3: Manual LAN / WLAN network access

c) Automated connection with the freeware plusoptiXconnect (only A12C and A16)

“plusoptiXconnect” is a freeware programme for automated connection and printing of measurement reports. device must be connected to the network via LAN / WLAN, too. The measurement reports stored on the SD card (A12C) or USB stick (A16) can now be portrayed, filtered and printed automatically on your workplace computer.

You can download the freeware and its user manual from our homepage (plusoptix.com), under support.

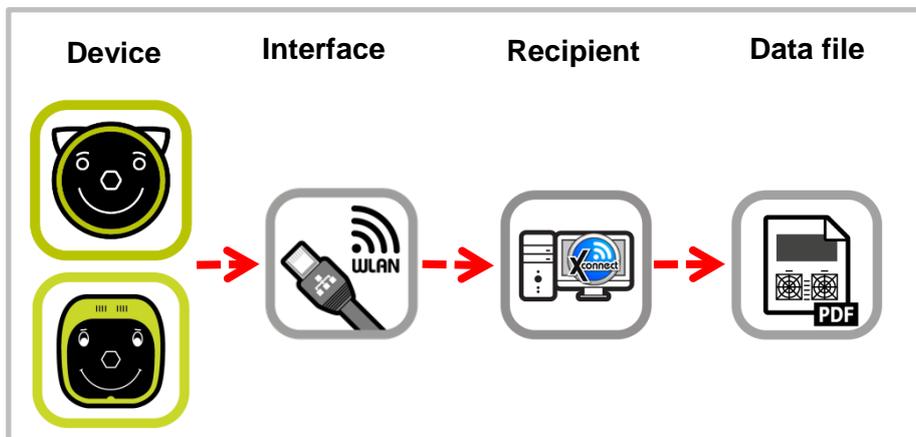


Figure 4: Automated connection with the freeware plusoptiXconnect

3 Full integration with your EMR system (only A12C and A16)

As an alternative to manual entry of the patient data, the “plusoptiX A12C” and “plusoptiX A16” can also be connected for a full integration (including automatic measurement report printing) with your EMR system.

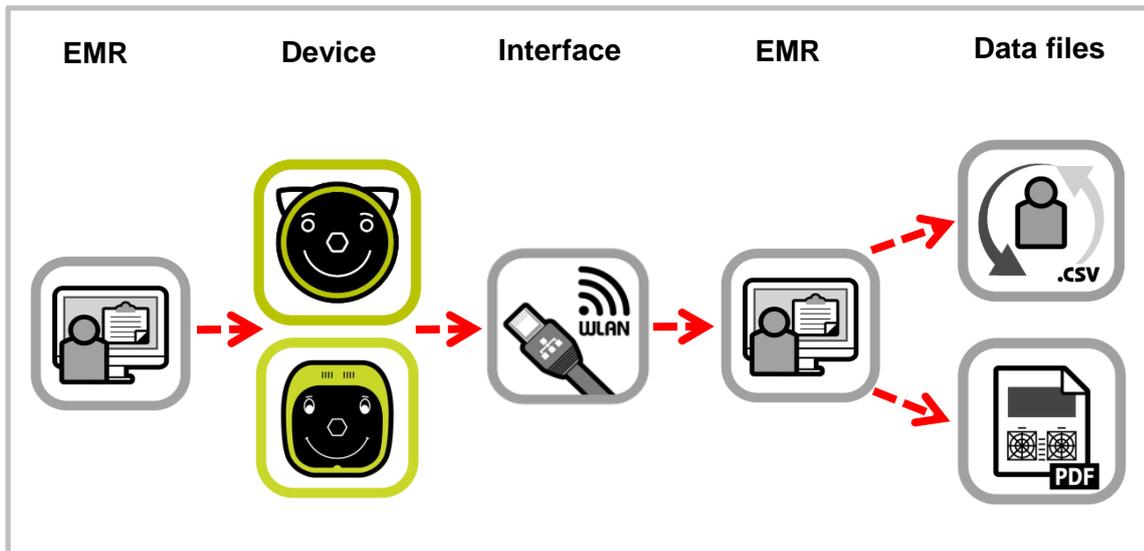


Figure 5: Automated connection to your Electronic Medical software

You simply have to choose a patient in your EMR system, and transfer the data to the device. After the measurement, the measurement results and the measurement report will be retransferred and saved to your EMR system.

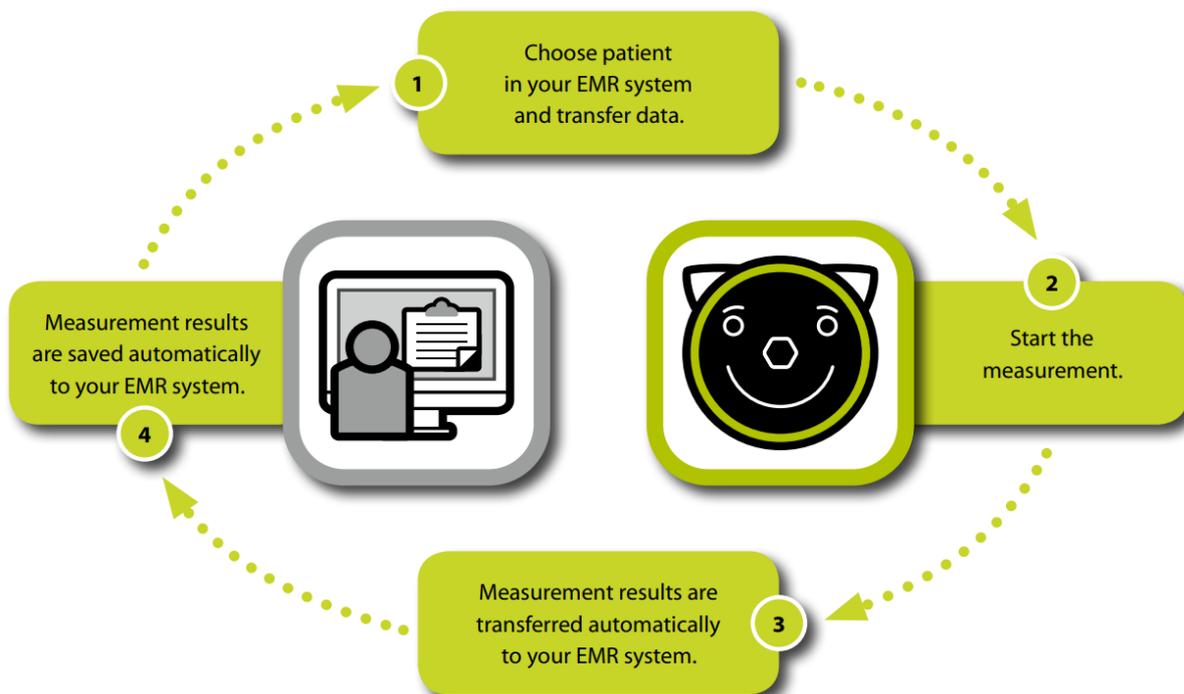


Figure 6: Automated transfer of patient data and measurement results

The measurement report can be printed on any printer that is connected with your network or workstation computer.

More information about csv file based EMR interface can be found on our homepage (plusoptix.com), under support.